

LOW FRICTION FABRIC UNDERGARMENTS for PRESSURE ULCER PREVENTION and MANAGEMENT ONLINE INFORMATION RESOURCES - PARAFRICTA.COM

Parafri^{ct}a®



Parafri^{ct}a®

Low friction fabric medical products

WEBSITE

STOP the Pressure Ulcer—*before it starts*

Please use the Site Contents menu at the top of the page to navigate our website



TRAINING VIDEO

0:05 / 8:55 • Introduction ▶



Parafri^{ct}a®

User Guide
For Undergarments


There are three styles of Parafri^{ct}a® undergarment.



Parafri^{ct}a® low friction fabric Undergarments are reusable products that are intended to aid in the prevention or healing of pressure ulcers, either alone or in combination with second dressings, moisture-wicking pads or pressure-relieving equipment. Pressure ulcers (Pressure Injuries/“bedsores”) can lead to serious and even life-threatening health consequences if left untreated. Always seek advice from a qualified healthcare professional if you think you, or a person in your care, has a pressure ulcer.

Instructions for Use

- Designed for continual wear under clothes or nightwear to protect skin on the sacrum, buttocks, and hips.
- May be used in combination with suitable pressure-relieving devices such as mattresses, cushions, or overlays.
- Select the appropriate style in the size according to the Size Selection table.
- If it is difficult for you or to get the slip-on styles on or off, consider using the Velcro® brand closure style, which can be opened at the front.



**Parafri^{ct}a® Low Friction Undergarments
FP10 Prescription Request**

Patient (NAME): _____ Date of Birth: _____

Health Centre: _____ Date: _____

Your Doctor / Nurse prescriber
Could you please prescribe the following items to assist in the treatment of the above patient.

PARAFRICTA® WASHABLE UNDERGARMENTS for pressure ulcer and skin damage relief

Velcro closure Brief Style	Product	Max Hip	Waist	Product Code	PIP Code	Drug	Number required
S	80	56.50		PI10G-02103	103-2103	136-85	
M	95	57.75		PI10G-02107	103-2107	136-85	
L	110	59.00		PI10G-02131	103-2131	136-85	
XL	125	60.25		PI10G-02155	103-2155	136-85	
XXL	140	61.50		PI10G-02179	103-2179	136-85	

Slip-on Brief Style	Product	Max Hip	Waist	Product Code	PIP Code	Drug	Number required
S	85	55.83		PI10G-02113	103-2113	136-85	
M	100	57.08		PI10G-02137	103-2137	136-85	
L	115	58.33		PI10G-02161	103-2161	136-85	
XL	130	59.58		PI10G-02185	103-2185	136-85	
XXL	145	60.83		PI10G-02209	103-2209	136-85	

Slip-on Boxer Style	Product	Max Hip	Waist	Product Code	PIP Code	Drug	Number required
S	95	56.67		PI10G-02119	103-2119	136-85	
M	110	57.92		PI10G-02143	103-2143	136-85	
L	125	59.17		PI10G-02167	103-2167	136-85	
XL	140	60.42		PI10G-02191	103-2191	136-85	
XXL	155	61.67		PI10G-02215	103-2215	136-85	

Use per item - Prices correct 01 December 2023

Client name: _____ Ref no: _____


Why is Parafri^{ct}a® being prescribed for the patient?

At risk of pressure ulcers Protect frail skin from damage

Existing pressure ulcer/skin damage Other _____

For further information please contact AFA Parafri^{ct}a Ltd on 01889 220566 or email info@parafri^{ct}a.com www.parafri^{ct}a.com

Signed: _____

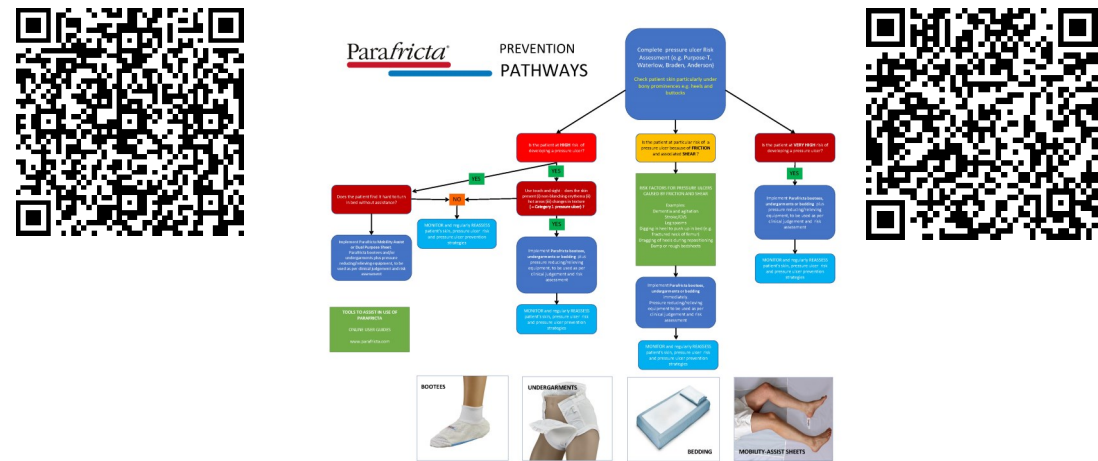

Parafri^{ct}a® PREVENTION PATHWAYS

Complete pressure ulcer Risk Assessment (e.g. Braden Q, Waterlow, Bixby, Addressall)

Check patient risk every 24 hours. Re-assess risk every 24 hours.

If the patient is at risk of developing a pressure ulcer:

- Use Parafri^{ct}a® Washable Undergarments (FP10) to reduce friction and shear.
- Use Parafri^{ct}a® Bedding (FP10) to reduce friction and shear.
- Use Parafri^{ct}a® Mobility Assist Sheets (FP10) to reduce friction and shear.

If the patient is at high risk of developing a pressure ulcer:

- Use Parafri^{ct}a® Washable Undergarments (FP10) to reduce friction and shear.
- Use Parafri^{ct}a® Bedding (FP10) to reduce friction and shear.
- Use Parafri^{ct}a® Mobility Assist Sheets (FP10) to reduce friction and shear.
- Use Parafri^{ct}a® Pressure Ulcer Prevention Pads (FP10) to reduce pressure.
- Use Parafri^{ct}a® Pressure Ulcer Prevention Cushions (FP10) to reduce pressure.
- Use Parafri^{ct}a® Pressure Ulcer Prevention Mattresses (FP10) to reduce pressure.

If the patient is at very high risk of developing a pressure ulcer:

- Use Parafri^{ct}a® Washable Undergarments (FP10) to reduce friction and shear.
- Use Parafri^{ct}a® Bedding (FP10) to reduce friction and shear.
- Use Parafri^{ct}a® Mobility Assist Sheets (FP10) to reduce friction and shear.
- Use Parafri^{ct}a® Pressure Ulcer Prevention Pads (FP10) to reduce pressure.
- Use Parafri^{ct}a® Pressure Ulcer Prevention Cushions (FP10) to reduce pressure.
- Use Parafri^{ct}a® Pressure Ulcer Prevention Mattresses (FP10) to reduce pressure.
- Use Parafri^{ct}a® Pressure Ulcer Prevention Dressings (FP10) to reduce pressure.

BOOTES **UNDERGARMENTS** **BEDDING** **MOBILITY ASSIST SHEETS**

Detailed Instructions for Use,
Size Selection, Washing Instructions

Information for prescribers and pharmacists

Pathways for pressure ulcer prevention